

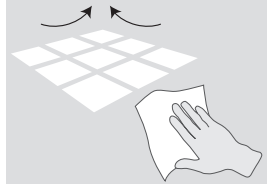
Alcohol-Free Surface Disinfectant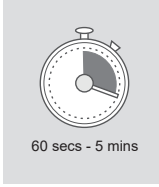

Specially formulated for use in the medical and dental environment. Contains detergent. For the surface disinfection of medical devices, including but not limited to, dental chairs, examination lights and patient transport. Not intended for use on medical and dental instruments.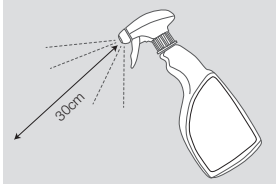

BPR For the disinfection of surfaces. Contains detergent. Use biocides safely and sustainably.

This product should be used in accordance with the product label. 5L Refill size may only be used to refill dispensers which are labelled with a DAFM approved label for the product. PT2 Formulation type: AL.

Directions for use: Spray clean surfaces and objects to be disinfected from a distance of 30cm so that they are completely moistened and let the disinfectant take action over 60 seconds (to achieve full efficacy leave for 5 minutes). Allow surface spray to dry or wipe with clean tissue. For professional use only.

Efficacy: Bactericidal (5 mins). Yeastical (5 mins). Virucidal activity against enveloped viruses (60 secs): Vaccinia virus, Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus, HIV, Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles, Poxviridae.

Tested to Standards: EN 14885, EN 14476, EN 13727, EN13697, EN 13624.

Composition: Contains Aqua, Alkyl (C12-16) dimethylbenzyl ammonium chloride (ADBAC/BKC (C12-16)) (68424-85-1) 0.48% w/w (4.8g/kg), Didecylidimethylammonium Chloride (7173-51-5) 0.22% w/w (2.2g/kg), Perfumes.

Warning: H412: Harmful to aquatic life with long lasting effects. **First aid procedures.** P301+P310: IF SWALLOWED: Immediately call a POISON CENTER or doctor. P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes.

Remove contact lenses, if present and easy to do. Continue rinsing. The wearing of protective gloves is recommended. UK number for poison advice: 111. Poisons Information Ireland: For information or to report a poisoning incident contact The National Poisons Information Centre (01 8092166). **Storage.** P403+P233: Store in a well-ventilated place. Keep container tightly closed. P235: Keep cool.

Contraindications. Incompatible materials: mild steel, aluminium, copper. **Disposal.** Dispose of contents/container in accordance with local regulations. P273: Avoid release to the environment. Do not discharge liquid into drains or rivers.

Efficacy

Microorganism	Time
Bactericidal	5 min
Pseudomonas aeruginosa	5 min
Staphylococcus aureus	5 min
Enterococcus hirae	5 min
Yeastical	5 min
Candida albicans	5 min
Enveloped viruses	60 sec
Vaccinia virus	60 sec
Hepatitis B virus (HBV)	60 sec
Hepatitis C virus (HCV)	60 sec
Hepatitis Delta virus	60 sec
HIV	60 sec
Influenza virus	60 sec
Coronavirus	60 sec
Herpesviridae	60 sec
Filoviridae (Ebola)	60 sec
Rubella virus	60 sec
Rabies virus	60 sec
Measles virus	60 sec
Poxviridae	60 sec

UFI: 2520-0057-K00H-5Q24

2022 NEW FORMULATION

Manufactured in the UK by: Topdental (Products) Ltd. 12 Ryefield Way, Silsden, W. Yorks BD20 0EF England, UK. Tel +44 (0)1535 652 750 www.topdental.co.uk

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