
UK

## Impression Disinfectant Powder

Fast acting disinfectant for all kinds of impression material such as silicone, alginates and polyether.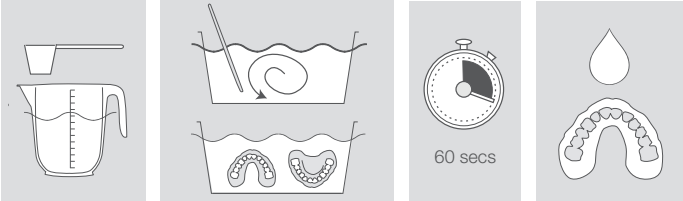

**Directions for use:** Dissolve 1 level measure of powder (20g ± 2g) in 980ml of warm tap water (30 - 40°C) and mix for 60 seconds. Place the impressions into the solution and fully immerse for the 60 second contact time. Do not exceed this recommended time. Rinse immediately with copious amounts of running water. Change solution daily. For professional use only.

**Efficacy:** Bactericidal (60 secs), including MRSA. Yeastcidal (60 secs). Viruses to include (60 secs): Non-enveloped viruses: Murine norovirus, Adenovirus 5. Enveloped viruses: Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus, Human Immunodeficiency virus (HIV), Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles. **Tested to Standards:** EN 14885, EN 14476, EN 13727, EN 13697, EN 13624.

**Composition:** TAED, Sodium Percarbonate, Citric Acid Monohydrate, Sodium Alkyl Aryl Sulphonate, Sodium Sulphate Anhydrous, Tetrasodium (1-hydroxyethylidene) bisphosphonate, Sodium Nitrite.

**Danger.** H318: Causes serious eye damage. H272: May intensify fire; oxidiser. H312: Harmful in contact with skin. P280: Wear protective gloves/protective clothing/eye protection/face protection. P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P301+P310: IF SWALLOWED: Immediately call a POISON CENTER or doctor. **Storage.** P403+P233: Store in a well-ventilated place. Keep container tightly closed. P235: Keep cool. P232: Protect from moisture. **Contraindications.** Incompatible materials: Aluminium, Copper. **Disposal.** Do not discharge into drains or rivers.

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