Instrument Disinfectant Powder with Colour change

Date of revision: 26/11/25

Available in 700g

UK

Instrument Disinfectant Powder with Colour change

Disinfectant powder for use in the reprocessing of all reusable surgical, medical and dental instruments prior to sterilisation. For use in the instrument soaking bath.

To Use: Dissolve 1 level measure of powder (20g ± 2g) in 980ml of warm tap water (30 - 40°C) and mix for 60 seconds or until the solution turns red. Completely immerse the instruments into the solution ensuring that no air is left within hollow instruments. Leave for 30 minutes. After disinfection, rinse instruments thoroughly with RO or distilled water prior to further processing and sterilisation. The solution will remain effective for 8 hours. Change to fresh solution daily or earlier should physical dirt in the solution become evident. We recommend preparing fresh solution at the start of the working day. When the solution is colourless it is no longer active. For professional use only.

Efficacy (30 mins): Bactericidal including MRSA. Yeastcidal. Sporicidal. Viruses to include: Non-enveloped viruses: Murine norovirus, Adenovirus 5. Enveloped viruses: Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus (HDV), HIV, RSV, Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles, Poxviridae.


Tested to Standards: EN17126, EN17111, EN14562, EN14561, EN14476, EN13727, EN13624.

Composition: Sodium Percarbonate, TAED, Citric Acid Monohydrate, Linear alkyl Benzene Sulphonate, Sodium Sulphate Anhydrous, Tetrasodium (1-hydroxyethylidene) bisphosphonate, Sodium Nitrite, Dye.

Contraindications. Aluminium, Copper. **Disposal.** Do not discharge into drains or rivers.

Danger. H318: Causes serious eye damage. H272: May intensify fire; oxidiser. H412: Harmful to aquatic life with long lasting effects. P280: Wear eye protection, protective gloves. P305+P351+P338+P310: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a doctor.

Storage. P210: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P220: Keep away from clothing and other combustible materials. Store in a well-ventilated place. Keep container tightly closed. Keep dry.

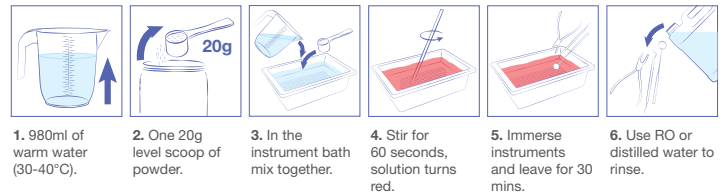1. 980ml of warm water (30-40°C).
2. One 20g level scoop of powder.
3. In the instrument bath mix together.
4. Stir for 60 seconds, solution turns red.
5. Immerse instruments and leave for 30 mins.
6. Use RO or distilled water to rinse.

Efficacy

Microorganism	Time
Bactericidal	30 min
<i>Pseudomonas aeruginosa</i>	30 min
<i>Staphylococcus aureus</i>	30 min
<i>Enterococcus hirae</i>	30 min
<i>Escherichia coli</i>	30 min
MRSA	30 min
Yeastcidal	30 min
<i>Candida albicans</i>	30 min
Enveloped viruses	30 min
Hepatitis B virus (HBV)	30 min
Hepatitis C virus (HCV)	30 min
Hepatitis Delta virus	30 min
HIV	30 min
RSV	30 min
Influenza virus	30 min
Coronavirus	30 min
Herpesviridae	30 min
Filoviridae (Ebola)	30 min
Rubella virus	30 min
Rabies virus	30 min
Measles virus	30 min
Poxviridae	30 min
Non enveloped viruses	30 min
Adenovirus 5	30 min
Murine norovirus	30 min
Sporicidal	30 min
<i>Bacillus subtilis</i>	30 min
<i>Bacillus cereus</i>	30 min

UK	Danger.
BG	Опасно.
CS	Nebezpečí.
DA	Fare.
DE	Gefahr.
EL	Κίνδυνος.
ES	Peligro.
ET	Ettevaatust.
FI	Vaara.
FR	Danger.
HR	Opasnost.
HU	Veszély.
IT	Pericolo.
LT	Pavojus.
LV	Bīstami.
NL	Gevaar.
NO	Fare.
PL	Niebezpieczeństwo.
PT	Perigo.
RO	Pericol.
SK	Nebezpečnosť.
SL	Nevarno.
SV	Fara.

Manufactured in the UK by:
Topdental (Products) Ltd.
Unit 1, Holmfild Industrial Estate,
Halifax, W. Yorks, HX2 9TN. UK
Tel +44 (0)1535 652750
www.topdental.co.uk

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