
# Instrument Disinfectant Powder with Colour change

Disinfectant powder for use in the reprocessing of all reusable surgical, medical and dental instruments prior to sterilisation. For use in the instrument soaking bath.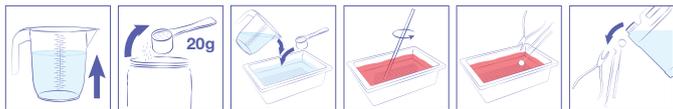

1. 980ml of warm water (30-40°C).
2. One 20g level scoop of powder.
3. In the instrument bath mix together.
4. Stir for 60 seconds, solution turns red.
5. Immerse instruments and leave for 30 mins.
6. Use RO or distilled water to rinse.

**Directions for Use:** Dissolve 1 level measure of powder (20g ± 2g) in 980ml of warm tap water (30 - 40°C) and mix for 60 seconds. The solution will turn red. Completely immerse the instruments into the solution ensuring that no air is left within hollow instruments. Leave for 30 minutes. After disinfection, rinse instruments thoroughly with RO or distilled water prior to further processing and sterilisation. The solution will remain effective for 8 hours. Change to fresh solution daily or earlier should physical dirt in the solution become evident. We recommend preparing fresh solution at the start of the working day. When the solution is colourless it is no longer active. For professional use only.

**Efficacy (30mins):** Bactericidal, including MRSA. Yeastcidal. Viruses to include: Non-enveloped viruses: Murine norovirus, Adenovirus 5. Enveloped viruses: Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus, Human Immunodeficiency Virus (HIV), Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles. Sporocidal.

**Tested to Standards:** EN 14885, EN 14476, EN 13727, EN 13704, EN 13697, EN 13624.

**Composition:** TAED, Sodium Percarbonate, Citric Acid Monohydrate, Sodium Alkyl Aryl Sulphonate, Sodium Sulphate Anhydrous, Tetrasodium (1-hydroxyethylidene) bisphosphonate, Sodium Nitrite, Dye.

**Danger.** H318: Causes serious eye damage. H272: May intensify fire; oxidiser. H312: Harmful in contact with skin. P280: Wear protective gloves/protective clothing/eye protection/face protection. P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P301+P310: IF SWALLOWED: Immediately call a POISON CENTER or doctor. **Storage.** P403+P233: Store in a well-ventilated place. Keep container tightly closed. P235: Keep cool. P232: Protect from moisture. **Contraindications.** Incompatible materials: Aluminium, Copper. **Disposal.** Do not discharge into drains or rivers.

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