

Alcohol Free Surface Disinfectant

Date of revision: 21/02/25

Available in 5 Litre / 2.5 Litre / 1 Litre & 500ml dispenser bottles


Intended Use / Intended Purpose:



For the surface disinfection of medical devices, including but not limited to, dental chairs, examination lights and patient transport. Not intended for use on medical and dental instruments. Specially formulated for use in the medical and dental environment.




For the disinfection of surfaces. Use biocides safely and sustainably. This product should be used in accordance with the product label. PT2 Formulation type: AL.

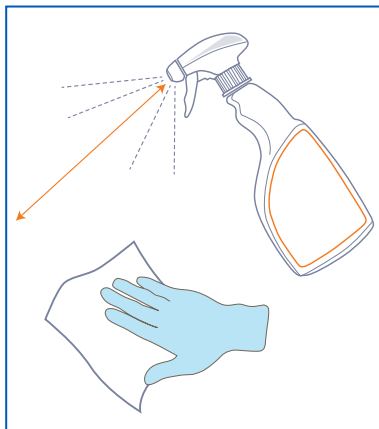

1. Spray clean surfaces and objects to be disinfected so that they are completely moistened and wipe with a clean cloth to distribute.



2. Let the disinfectant take action for 60 seconds.

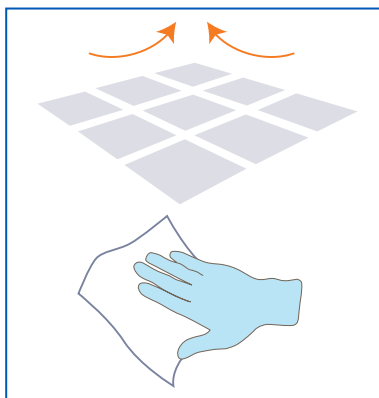

3. Allow surface moisture to air dry or wipe with clean tissue. See product labels and MSDS for further information.

Indications / Efficacy (60 seconds): Bactericidal. Yeasticidal. Virucidal activity against enveloped viruses: Vaccinia virus, Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus, HIV, Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles, Poxviridae.

Tested to Standards: EN16777, EN16615, EN14885, EN14476, EN13727, EN13624.

Composition: Contains Aqua, Alkyl (C12-16) dimethylbenzyl ammonium chloride (ADBAC/BKC (C12-16)) (68424-85-1) 0.48% w/w (4.8g/kg), Didecyldimethylammonium Chloride (7173-51-5) 0.22% w/w (2.2g/kg), Perfumes (contain Amyl cinnamal, Limonene, Alpha-Isomethyl ionone).

Handling and Storage. Ensure there is sufficient ventilation of the area. The wearing of protective gloves is recommended. P403+P233: Store in a well-ventilated place. Keep container tightly closed. P235: Keep cool.

Contraindications. Mild steel, aluminium, copper.

Disposal. Dispose of contents/container in accordance with local regulations. Do not discharge liquid into drains or rivers.

Intended Users. For professional use only. The products are used within the dental surgery by the dentist or dental assistant as part of their general disinfecting routines.

Patient Population: Dental surgery patients, both adults and children.

Contains detergent. UK number for poison advice: 111. Poisons Information Ireland: For information or to report a poisoning incident contact The National Poisons Information Centre (01 8092166). 5L Refill size may only be used to refill dispensers which are labelled with a DAFM approved label for the product.

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

H412: Harmful to aquatic life with long lasting effects.
P273: Avoid release to the environment. P301+P310:
IF SWALLOWED: Immediately call a doctor.
P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.



For additional languages
see www.topdental.co.uk



Manufactured in the UK by:
Topdental (Products) Ltd.
Unit 1, Holmfield Industrial Estate,
Halifax, W. Yorks, HX2 9TN. UK
Tel +44 (0)1535 652750
www.topdental.co.uk



Authorisation holder: DIRECTA
AB P.O. Box 723, Finvids väg
8 SE-194 27 Upplands Väsby,
Sweden



UFI: 2520-0057-K00H-5Q24