


## Daily Aspirator Cleaner and Disinfectant

Date of revision: 17/01/25


Available in 5 Litre / 2.5 Litre / 1 Litre dispenser bottle

**Intended Use / Intended Purpose:** For the daily cleaning and maintenance of Dental Aspirator Systems.  
Can be used on wet line, dry line and semi wet/dry line suction systems

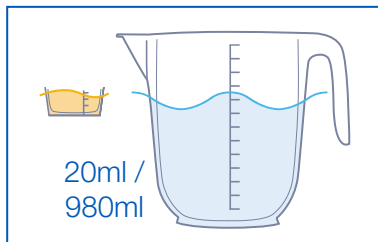

1. Add 20ml  $\pm$  1ml of concentrate to 980ml warm water (2% solution).


The 1L bottle has a built in dosing device and the 2.5L a dosing cap.

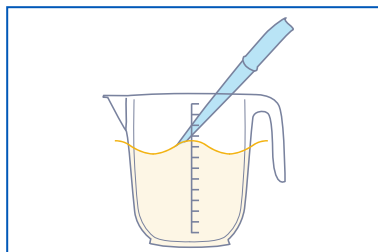


2. Aspirate via the high and low volume suction tubes.



3. Leave for a minimum of 15 minutes.

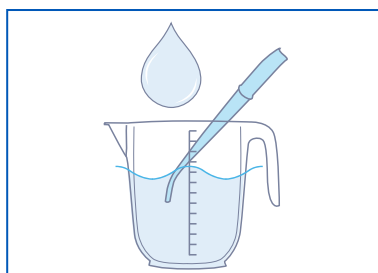

4. Rinse thoroughly with clean water.

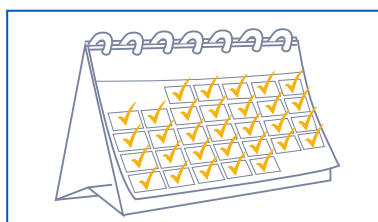

5. Use daily. See product labels and MSDS for further information.

**Indications / Efficacy:** (15 mins) Bactericidal. Yeasticidal. Viruses to include: Non-enveloped viruses: Murine norovirus, Adenovirus 5. Enveloped viruses: Hepatitis B virus, Hepatitis C virus, Hepatitis Delta virus, HIV, Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles, Poxviridae.

**Tested to Standards:** EN17111, EN14885, EN14562, EN14561, EN 14476, EN13727, EN 13624.

**Composition:** Aqua, Dimethyldioctylammonium chloride, Dipropylene glycol, 2-Aminoethanol, Tetrasodium ethylene diamine tetraacetate, Ethanol, Citric acid, Nonionic surfactant, Antifoam, Dye.

**Storage.** P403+P233: Store in a well-ventilated place. Keep container tightly closed. P235: Keep cool. **Contraindications.** Aluminium.

**Disposal.** Do not discharge concentrate into drains or rivers. IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. IF INHALED: Remove person to fresh air and keep comfortable for breathing.

**Intended Users.** For professional use only. The products are used within the dental surgery by the dentist or dental assistant as part of their general disinfecting routines.

**Patient Population:** Dental surgery patients, both adults and children.

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

**Danger.** H315: Causes skin irritation. H318: Causes serious eye damage. H410: Very toxic to aquatic life with long lasting effects. P280: Wear protective gloves, protective clothing, eye protection, face protection. P302+P352: IF ON SKIN: Wash with plenty of water. P305+P351+P338+P310: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a doctor. P332+P313: If skin irritation occurs: Get medical advice/attention. P391: Collect spillage.



**Danger.**



For additional languages see [www.topdental.co.uk](http://www.topdental.co.uk)



Manufactured in the UK by:  
Topdental (Products) Ltd.  
Unit 1, Holmfield Industrial Estate,  
Halifax, W. Yorks, HX2 9TN. UK  
Tel +44 (0)1535 652750  
[www.topdental.co.uk](http://www.topdental.co.uk)



DIRECTA AB P.O. Box 723,  
Finvids väg 8 SE-194 27  
Upplands Väsby, Sweden



UFI: J110-E062-M003-J8G9