


3 in 1 Disposable Air/Water Syringe Tips - Spectrum Tips

Date of revision: 29/10/24


Available in resealable packs of 250 tips

This guide is intended for doctors and dental staff. Before using the 3 in 1 Disposable Air/Water Syringe Tips check the label for any cautions, warnings and storage instructions.

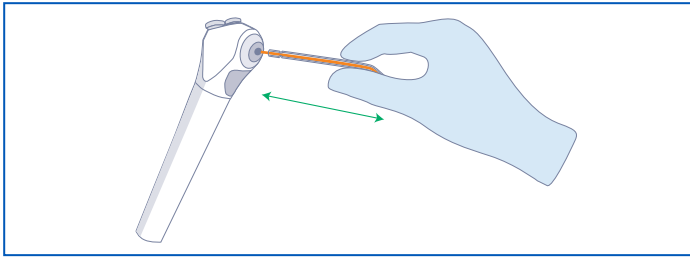


1. INSTALL THE TIP

Push the syringe tip into the syringe tip retainer assembly until you feel two clicks. **WARNING - If you do not feel two clicks as you install the syringe tip, do not use the syringe. Check the syringe and refer to manufacturers instructions.**

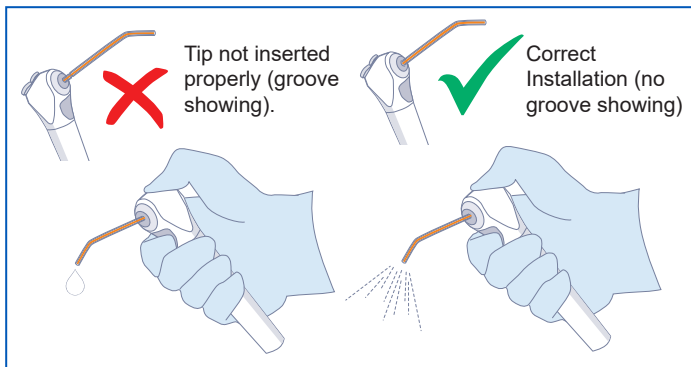

2. CHECK THE TIP IS INSTALLED CORRECTLY

Visual check: Complete Installation (no groove showing), Incomplete Tip Installation (groove showing). Manual check: Point the syringe at the floor and press the air button several times to be sure the tip is properly installed. Express water then express air only, if air contains moisture refer to syringe manufacturers instructions.

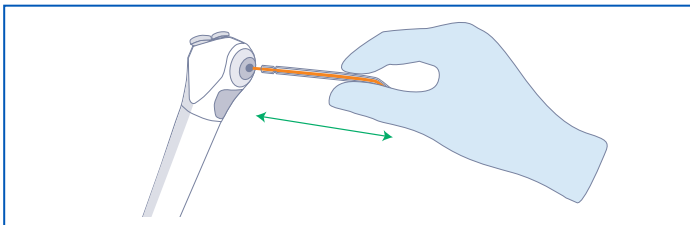

3. AFTER USE

Remove the syringe tip by pulling the tip straight out of the syringe and discard.

Intended Use / Intended Purpose: Used to supply water, compressed air or mist into the oral cavity for the purpose of irrigation, cleaning debris away from the area or individual tooth work (drying). Used in dental procedures as a single-use, disposable device to prevent cross-contamination between patients.

Intended Users. For professional use only. The products are used within the dental surgery by the dentist or dental assistant.

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

Indications: Used in dental procedures as a single-use, disposable device to prevent cross-contamination between patients.

Performance Characteristics: • Separate air and water channels to prevent cross over, delivers dry air. • Secure fitting. • Prevents cross-contamination between patients. • Bevelled edges for patient safety.

Dental Units/Syringe Types: Spectrum Tips can be used in syringes that have internal locking retainers.

Disposal: Dispose of in accordance with local waste disposal guidelines/regulations.

Patient Population: Dental surgery patients, both adults and children.

Contraindication: 3 in 1 Disposable Air/Water Syringe Tips are for single use only. Re-use may result in cross contamination.

WARNING

Ensure syringe tips are correctly inserted and locked in place. Using syringe tips not secured in place may result in syringe tip ejection. Tip ejection may result in ingestion or aspiration of the tip, requiring immediate medical attention.



Manufactured in the UK by:
Topdental (Products) Ltd.
Unit 1, Holmfield Industrial Estate,
Halifax, W. Yorks, HX2 9TN. UK
Tel +44 (0)1535 652750
www.topdental.co.uk



DIRECTA AB P.O. Box 723,
Finvids väg 8 SE-194 27
Upplands Väsby, Sweden

