



DAILYasp

Aspirator Cleaner & Disinfectant

EN14476
VIRUS TESTED

2%
DILUTION
20ml/L*

CONTACT
15 MIN*

LOW
FOAMING

SAFE FOR
ALL TYPES

KEY FEATURES

Bossklein DAILYasp has been designed specifically for daily use to remove contamination and micro-organisms from inside aspirator suction systems to help prevent the transmission of disease.

In order to reduce the risk of infection for patients it is important that cleaning and disinfection of these type of machines takes place regularly.

DAILYasp has been developed for the daily cleaning and disinfection all types of dental aspirator suction systems. Including wet line, dry line and semi wet/dry systems.

The low-foaming and non-abrasive formulation ensures maximum microbial effect with no risk to sensitive equipment. The aldehyde, chlorine and phenol free solution is easy and safe to use.

- Short soaking time from only 15 mins
- Virus tested to EN 14476
- Effective against TB
- Economical concentrate (2%)
- Low foaming & non abrasive
- Aldehyde, Chlorine & Phenol free
- 3 in 1 – Cleans, disinfects and deodorises

The highly effective solution is Bactericidal, Yeastcidal and Tuberculocidal, ensuring this one of the most effective solutions available on the market today. It is also effective against Hepatitis B, Hepatitis C and HIV viruses.

COMPOSITION

Aqua, Dimethyldioctylammonium chloride, Dipropylene glycol, 2-Aminoethanol, Tetrasodium ethylene diamine tetraacetate, Ethanol, Citric acid, Nonionic surfactant, Antifoam, Dye.

INSTRUCTIONS FOR USE

Add 20ml ± 1ml of concentrate to 980ml warm water (2% solution). Aspirate via the high and low volume suction tubes. Leave for a minimum of 15 minutes (to achieve full efficacy leave for 60 minutes) and then rinse thoroughly with clean water. For professional use only.

PHYSICAL PROPERTIES

Liquid
Biodegradable
pH: 11.0-12.0
36 month shelf life from date of manufacture

TEST STANDARDS

The following test standards have been achieved:
EN14885, EN14476, EN14348, EN13727, EN13697, EN13624, ASTM E1052.

VARIANTS AND PACKAGING

BOS2487 - 1 Litre x 6 per box
BOS2489 - 2.5 Litre x 4 per box
BOS2488 - 5 Litre x 2 per box

MICROBIOLOGICAL EFFICACY

MICROORGANISM	TIME	CONC %
Bactericidal	15 min	2%
BS EN 13727:2012 + A1:2013		
BS EN 13697:2015		
Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae, Salmonella typhimurium, Legionella pneumophila	15 min	2%
Yeastcidal	15 min	2%
BS EN 13624:2013		
Candida Albicans	15 min	2%
Viruses		
BS EN 14476 2013 + A1 2015		
ASTM E1052 - 96		
Hepatitis B (HBV), Hepatitis C (HCV), HIV	15 min	2%
Fungicidal	30 min	2%
BS EN 13624:2013		
Aspergillus brasiliensis	30 min	2%
Tuberculocidal	60 min	2%
BS EN 14348:2005		
Mycobacterium terrae	60 min	2%

* For full efficacy and specification see Bossklein.com

