











Decontamination

A quick guide

Bossklein®

Instruments	 <div>IDactiv Instrument Disinfectant and Cleaner</div>	For use inside a soaking or ultrasonic bath to clean manual instruments inc scalpels, forceps, luxators, etc.	Dilution: 2%-20^{ml}/1L	Contact Time: 15^{min}	<div>Bactericidal15 min2%</div> <div>Yeasticidal15 min2%</div> <div>All Enveloped Viruses15 min2%</div> <div>Non-Enveloped Viruses Adenovirus 5, Norovirus30 min4%</div> <div>Mycobactericidal60 min4%</div> <div>Tuberculocidal60 min4%</div>
Instruments	 <div>IDactiv QUANTUM Sporicidal Instrument Disinfectant and Cleaner</div>	For use inside a soaking bath to clean manual instruments inc scalpels, forceps, luxators, etc.	Dilution: 2%-20^g/1L	Contact Time: 30^{min}	<div>Bactericidal30 min2%</div> <div>Yeasticidal30 min2%</div> <div>All Enveloped Viruses30 min2%</div> <div>Non-Enveloped Viruses Adenovirus 5, Norovirus30 min2%</div> <div>Sporidical30 min2%</div>
Instruments	 <div>ULTRA bio Ultrasonic Instrument Cleaner</div>	For use inside an ultrasonic bath to clean manual instruments inc scalpels, forceps, luxators, etc.	Dilution: 3%-30^{ml}/1L	Contact Time: 15^{min}	<div>Cleaning solution only. Includes enzymes for powerful cleaning action. Not recommended for plastics and rubbers.</div>
Surfaces	 <div>V-WIPE Surface Disinfectant Wipes with Alcohol</div>	For use in clinical areas on alcohol resistant surfaces inc worktops, inspection lamps, hand controls, etc.	Dilution: READY TO USE	Contact Time: 60^{sec}	<div>Bactericidal60 sec</div> <div>Yeasticidal60 sec</div> <div>All Enveloped Viruses60 sec</div> <div>Non-Enveloped Viruses Adenovirus 5, Norovirus60 sec</div> <div>Mycobactericidal60 sec</div> <div>Tuberculocidal60 sec</div>
Surfaces	 <div>SDactiv Surface Disinfectant Spray with Alcohol</div>	For use in clinical areas on alcohol resistant surfaces inc worktops, inspection lamps, hand controls, etc.	Dilution: READY TO USE	Contact Time: 60^{sec}	<div>Bactericidal60 sec</div> <div>Yeasticidal60 sec</div> <div>All Enveloped Viruses60 sec</div> <div>Non-Enveloped Viruses Adenovirus 5, Norovirus60 sec</div> <div>Mycobactericidal60 sec</div> <div>Tuberculocidal60 sec</div>
Surfaces	 <div>V-WIPE ZERO Alcohol Free Surface Disinfectant Wipes</div>	For use in clinical areas on sensitive surfaces inc the dental chair* Contains detergent for added cleaning power.	Dilution: READY TO USE	Contact Time: 60^{sec}	<div>Bactericidal60 sec</div> <div>Yeasticidal60 sec</div> <div>All Enveloped Viruses60 sec</div> <div>Including: Vaccinia, Hepatitis, HIV, Measles, Influenza, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Poxviridae</div>
Surfaces	 <div>SDactiv ZERO Alcohol Free Surface Disinfectant Spray</div>	For use in clinical areas on sensitive surfaces inc the dental chair* Contains detergent for added cleaning power.	Dilution: READY TO USE	Contact Time: 60^{sec}	<div>Bactericidal60 sec</div> <div>Yeasticidal60 sec</div> <div>All Enveloped Viruses60 sec</div> <div>Including: Vaccinia, Hepatitis, HIV, Measles, Influenza, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Poxviridae</div>
Surfaces	 <div>V-WIPE ZERO ECO-XL Plastic and Alcohol Free Surface Disinfectant Wipes</div>	For use in clinical areas on sensitive surfaces inc the dental chair* Contains detergent for added cleaning power.	Dilution: READY TO USE	Contact Time: 60^{sec}	<div>Bactericidal2 min</div> <div>Yeasticidal60 sec</div> <div>All Enveloped Viruses60 sec</div> <div>Including: Vaccinia, Hepatitis, HIV, Measles, Influenza, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Poxviridae</div>
Special Areas	 <div>DAILY asp Aspirator Disinfectant and Cleaner</div>	For everyday use with all types of dental aspirator / suction systems.	Dilution: 2%-20^{ml}/1L	Contact Time: 15^{min}	<div>Bactericidal15 min2%</div> <div>Yeasticidal15 min2%</div> <div>All Enveloped Viruses15 min2%</div> <div>Including: Vaccinia, Hepatitis, HIV, Measles, Corona, + more</div> <div>Non-Enveloped Viruses Adenovirus 5, Norovirus15 min2%</div>
Special Areas	 <div>IMPactiv P Impression Disinfectant Powder</div>	For use with all types of dental impression materials including orthodontic impressions.	Dilution: 2%-20^g/1L	Contact Time: 60^{sec}	<div>Bactericidal60 sec2%</div> <div>Yeasticidal60 sec2%</div> <div>All Enveloped Viruses60 sec2%</div> <div>Including: Vaccinia, Hepatitis, HIV, Measles, Corona, + more</div> <div>Non-Enveloped Viruses Adenovirus 5, Norovirus60 sec2%</div>