

# Weekly Aspirator Cleaner and Disinfectant

# Bossklein®



## KEY FEATURES

For the weekly cleaning and disinfection of all types of aspirator/suction systems. Can be used on wet line, dry line and semi wet/dry line systems.

- Contact time of 30 minutes
- Low hazard classification
- Acid based
- Fully Bactericidal
- Green in colour
- Concentrate solution
- 5% dilution ratio

Bossklein Weekly Aspirator is free from chlorides, aldehyde and phenols.

The low foaming and non abrasive formulation ensures equipment remains unaffected by the solution.

The highly effective solution is fully Bactericidal within 30 minutes.

## COMPOSITION

Contains citric acid, benzalkonium chloride, non-ionic surfactant, antifoam, dyestuff.

## INSTRUCTIONS FOR USE:

Add 50ml ± 1ml of the concentrate to 950ml of warm water (5% solution). Aspirate via the high and low volume suction tubes. Leave for a minimum of 30 minutes and then rinse thoroughly with clean water. Do not use on aluminium or copper parts. For professional use only.

## TEST STANDARDS:

The following test standards have been achieved: EN 14885, EN 13727.

## PHYSICAL PROPERTIES:

Green liquid with a mild odour  
Biodegradable  
pH: 1.50-2.50  
36 month shelf life from date of manufacture

## VARIANTS AND PACKAGING:

BOS9806 - 1 Litre x 6 per box  
BOS9809 - 5 Litre x 2 per box

## MICROBIOLOGICAL EFFICACY

MICRO-ORGANISM	CONC %	TIME
Bactericidal	5% 50ml/1L	30 mins
Pseudomonas aeruginosa	5% 50ml/1L	30 mins
Staphylococcus aureus	5% 50ml/1L	30 mins
Enterococcus hirae	5% 50ml/1L	30 mins



Complete **hygiene** solutions

DirectaDentalGroup  
DIRECTA topdental ORSING parkell RONVIG TröllDental

Topdental (Products) Ltd. 12 Ryefield Way, Silsden, West Yorkshire, BD20 0EF, England UK.  
Tel +44 (0)1535 652 750 www.topdental.co.uk

v1.1.2005